Checkliste Übergabe digipaX 2 / Handover checklist digipaX 2

1 Allgemeines / General

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This document is an internal checklist for distributors and administrators, to be included with the product digipaX 2 to the final customer.

A **server** is referred to as the station on which image archive is installed and which will normally be used for image acquisition. **Workstations** are determined to be all the other computers on the network, on which staff works within the system.

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The following precautions have been taken to prevent hazards in the event of a system failure (if necessary):

	performed
▪ Alternative computers can be used	
▪ Server can be used as a workstation	

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The following precautions have been taken to prevent loss of data:

	performed
▪ Use of RAID system	
▪ Automatic data mirroring to another hard disk or backups system	
▪ Installation and configuration of a tool for securing data on CD/DVD (weekly/every 14 days)	

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	performed
For the finally installed system, tests should be executed according to the document „Acceptance test at the cus-	

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toomer digipaX 2“ and checked off or mark as NA (not applicable) if the corresponding functions are not installed.	
--	--

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	performed
Installation protocol has been completed.	

In this protocol, updates installed later at each client should be added. It must be possible to determine the clients of the specific version of the product at any time.

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	performed
▪ Securing the digipaX system path (dpxsys)	

The settings are important so that later the processes, which are dependent on the settings, can be tracked from outside the system. In order to secure the configurations, copy the full digipaX system path (dpxsys).

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	performed
▪ Adjustment of the organ chart proceeded	
▪ Explanation of workflow	
▪ Explanation of image selection for components with drag & drop and right-click (quick navigation)	
▪ Explanation of creating and deleting worklist jobs (visually and textually) and the selection of the patients from the worklist	
▪ When using third-party systems via DICOM Worklist with procedure codes: <ul style="list-style-type: none"> ○ test the system's response in the absence of a code and suitable user in- 	

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<ul style="list-style-type: none"> o test the system's response for unknown procedure code and suitable user instruction o determine how to check the system if the code matches the image acquisition (for example, display a plain text description) 	
<ul style="list-style-type: none"> ▪ Explanation call up the quick tools with a right-click and explanation of adding and removing tools 	
<ul style="list-style-type: none"> ▪ Explanation of windowing function 	
<ul style="list-style-type: none"> ▪ Explanation of filter functions (viewer) 	
<ul style="list-style-type: none"> ▪ Advice, which you should pay special attention to, so that upon receipt of data from third-party systems (patient CDs, transfer of received images) match the target patient data -> avoid the mismatching of data 	
<ul style="list-style-type: none"> ▪ Guidelines for the environment conditions for diagnosing 	
<ul style="list-style-type: none"> ▪ Explanation of the daily visual inspection 	
<ul style="list-style-type: none"> ▪ Explanation of data securing and spacial separating for the weekly data securing on CD/DVD + Note on storage and life time of the media (observe manufacturer instructions) 	
<ul style="list-style-type: none"> ▪ Advice to compliance of the user manual <ul style="list-style-type: none"> o Explanation and demonstration of calling of the PDF user manual o Contact information of the manufacturer to last page of the manual o Free user manual on paper can be delivered within 5 working days 	

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	performed
<ul style="list-style-type: none"> ▪ Checking whether the description of the current recording matches the order 	<input type="checkbox"/>
<ul style="list-style-type: none"> ▪ Explanation of generator control when generator is directly connected 	<input type="checkbox"/>
<ul style="list-style-type: none"> ▪ In the case of detectors with a rechargeable battery, indication of the level indicator and timely recharging 	<input type="checkbox"/>
<ul style="list-style-type: none"> ▪ When taking the image before triggering the radiation: <ul style="list-style-type: none"> ○ Check if correct target detector is activated (when using multiple detectors) ○ wait for readiness display ("Ready for acquisition") 	<input type="checkbox"/>
<ul style="list-style-type: none"> ▪ Explanation of the image acquisition process including backup/restore function 	<input type="checkbox"/>
<ul style="list-style-type: none"> ▪ Explanation Image cropping function 	<input type="checkbox"/>
<ul style="list-style-type: none"> ▪ Explanation of how to change the image filter 	<input type="checkbox"/>
<ul style="list-style-type: none"> ▪ Explanation Exposure index and Deviation index + checking the dose indicator mask before drawing conclusions from the dose indicator 	<input type="checkbox"/>