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A **server** is referred to as the station on which image archive is installed and which will normally be used for image acquisition. **Workstations** are determined to be all the other computers on the network, on which staff works within the system.

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- the appropriate threshold value for the archive fill state + correct indication on all stations
- storage location for X-ray journal -> RAID system
- checking the number of backups for image acquisition (as high as possible – about 50 acquisitions)

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Restart the server and workstation, on which the test is carried out!

Perform and check off the following test steps:

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Test instruction	performed successfully
If the GDT/HL7 connection is made, transfer through each system by GDT/HL7 data transfer of patient data to digipaX or even requesting creation of a new job <ul style="list-style-type: none"> <li>➔ depending on what settings in the systems were possible</li> <li>➔ use test patients with diacritical characters (if existing in user`s language)</li> </ul>	
Create any job on the workstation for any patient <ul style="list-style-type: none"> <li>➔ job for the image</li> </ul>	

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Reload this job on the server, compare the patient name and the description of the job → must match the assumed data	
Download a test image and send to the image archive → succesful image acquisition and no errors when saving	
On the workstation reload this patient and download the image → image must be downloaded and displayed correctly	
If the GDT/HL7 response was activated, also check additionally whether the response was created and succesfullt processed by the recipient → e.g. entry into index card	