## Checkliste Übergabe / Handover checklist

### 1 Allgemeines / General

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This document is an internal checklist for distributors and administrators, to be included with the product digipaX to the final customer.

A **server** is referred to as the station on which image archive is installed and which will normally be used for image acquisition. **Workstations** are determined to be all the other computers on the network, on which staff works within the system.

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The following precautions have been taken to prevent loss of data:

	performed
▪ Use of RAID system	
▪ Automatic data mirroring to another hard disk or backups system	
▪ Installation and configuration of a tool for securing data on CD/DVD (weekly/every 14 days)	

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	performed
For the finally installed system, tests should be executed according to the document „Acceptance test at the customer“ and checked off or mark as NA (not applicable) if the corresponding functions are not installed.	

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performed
Installation protocol has been completed.

In this protocol, updates installed later at each client should be added. It must be possible to determine the clients of the specific version of the product at any time.

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The settings are important so that later the processes, which are dependent on the settings, can be tracked from outside the system. In order to secure the configurations, perform the following steps:

- open digipaX
- activate the configuration
- log in as administrator
- switch „Export...“
- select all and OK
- select and save location and file names
- additionally secure the files vet2.ini, HL7.ini and display.xml from the digipaX installation directory (typically: C:\digipaX\digipaX)

performed
▪ Securing the server configuration
▪ Securing the workstation configuration

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performed
▪ Adjustment of the organ chart proceeded
▪ Explanation of workflow
▪ Explanation of creating and deleting worklist jobs (visually and textually) and the selection of the

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patients from the worklist	
<ul style="list-style-type: none"> <li>When using third-party systems via DICOM Worklist with procedure codes: <ul style="list-style-type: none"> <li>test the system's response in the absence of a code and suitable user instruction</li> <li>test the system's response for unknown procedure code and suitable user instruction</li> <li>determine how to check the system if the code matches the image acquisition (for example, display a plain text description)</li> </ul> </li> </ul>	
<ul style="list-style-type: none"> <li>Explanation when capturing images using Perkin Elmer XRpad: <ul style="list-style-type: none"> <li>wait for green ready message before doing the exposure</li> <li>determining the state of the battery when using the WiFi panel on the basis of the symbol in the software</li> <li>check exposure index mask before doing conclusions</li> <li>when using multiple detectors: determining activated detector and change detector if needed before doing the exposure</li> </ul> </li> </ul>	
<ul style="list-style-type: none"> <li>Explanation of the image acquisition functions (XRay, MRI, framegrabber / video, special image series acquisition), including backup/restore functions and cropping the image, if needed</li> </ul>	
<ul style="list-style-type: none"> <li>Explanation of image processing filters for MRI image acquisition</li> </ul>	
<ul style="list-style-type: none"> <li>Explanation of windowing function</li> </ul>	
<ul style="list-style-type: none"> <li>Explanation of filter functions (viewer)</li> </ul>	
<ul style="list-style-type: none"> <li>Advice, which you should pay special attention to, so that upon receipt of data from third-party systems (patient CDs, transfer of received</li> </ul>	

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images) match the target patient data -> avoid the mismatching of data	
▪ Guidelines for the environment conditions for diagnosing	
▪ Explanation of the daily visual inspection	
▪ Explanation of data securing and spacial separating for the weekly data securing on CD/DVD + Note on storage and life time of the media (observe manufacturer instructions)	
▪ Advice to compliance of the user manual <ul style="list-style-type: none"> <li>○ Explanation and demonstration of calling of the PDF user manual</li> <li>○ Contact information of the manufacturer to last page of the manual</li> <li>○ Free user manual on paper can be delivered within 5 working days</li> </ul>	